



PREGNANCY TEST DIPSTICK WORKSHEET

5 simple steps to a perfect result

TEST PROCEDURE nb: use of early morning urine (EMU) is strongly recommended


1

Bring the test Dipstick and the sample to room temperature (15°-30°C).



2

Insert the Dipstick into the sample
DO NOT immerse past the MAX line.

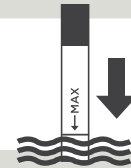

3

Hold in the sample for **10 seconds** then lay the Dipstick on a clean, level surface and wait.



4

Read the results 5 minutes after dip.
DO NOT READ BEFORE 5 MINUTES.



INTERPRETATION PROCEDURE

5

RESULT: POSITIVE

If two red colour bands appear, one in each zone, this is a POSITIVE result and indicates the hCG level is at or above the detection sensitivity.

RESULT: POSITIVE

Any line in the test zone, no matter how faint, is a POSITIVE result.

RESULT: NEGATIVE


If one red colour band appears in the control zone with none in the test zone this result is NEGATIVE.

RESULT: INVALID

If no bands appear the result is INVALID - retry, making sure you use the correct test procedure and check the Dipstick is not out of date or otherwise compromised.

RESULT: INVALID

If a band only appears in the test zone the result is INVALID - retry, making sure you use the correct test procedure and check the Dipstick is not out of date or otherwise compromised.

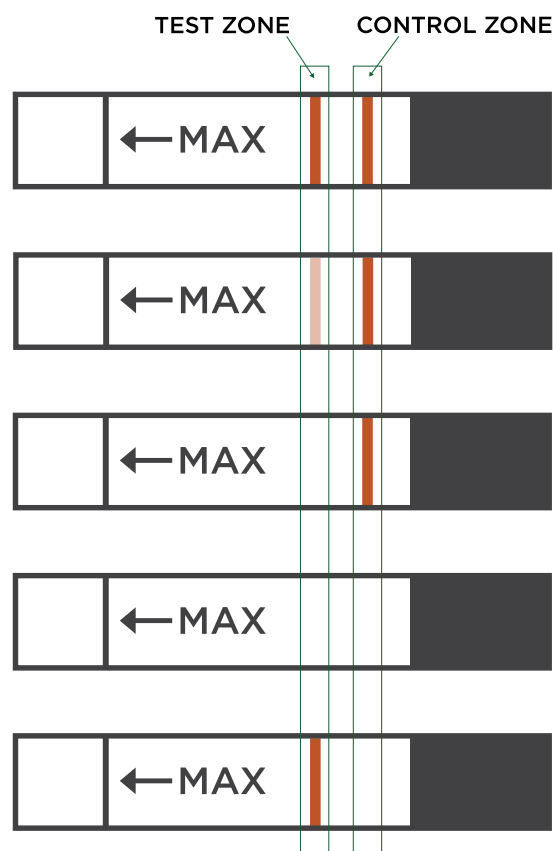

IMPORTANT: Please read the kit instructions for full information, including the limitations of urine pregnancy tests.