



PREGNANCY TEST CASSETTE WORKSHEET

5 simple steps to a perfect result

TEST PROCEDURE

nb: use of early morning urine (EMU) is *strongly* recommended

1

Bring the test Cassette and the sample to room temperature (15°-30°C).



2

Lay the test Cassette on a clean, level surface. Load the supplied Pipette with patient urine.



3

Add 3 to 4 drops of urine to the sample well.



4

Read results 5 minutes after adding sample.
DO NOT READ BEFORE 5 MINUTES.



INTERPRETATION PROCEDURE

5

RESULT: POSITIVE

If two red colour bands appear, one in each zone, this is a POSITIVE result and indicates the hCG level is at or above the detection sensitivity.

RESULT: POSITIVE

Any line in the test zone, no matter how faint, is a POSITIVE result.

RESULT: NEGATIVE

If one red colour band appears in the control zone with none in the test zone this result is NEGATIVE.





RESULT: INVALID

If no bands appear the result is INVALID - retry, making sure you use the correct test procedure and check the Cassette is not out of date or otherwise compromised.

RESULT: INVALID

If a band only appears in the test zone the result is INVALID - retry, making sure you use the correct test procedure and check the Cassette is not out of date or otherwise compromised.

CONTROL ZONE TEST ZONE

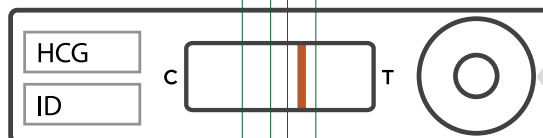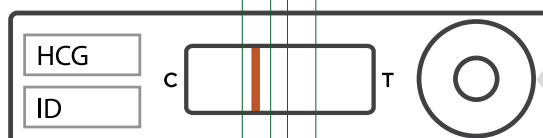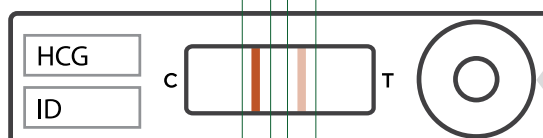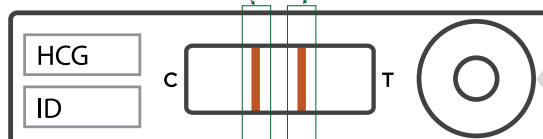

IMPORTANT: Please read the kit instructions for full information, including the limitations of urine pregnancy tests.