

TEST PROCEDURE FOR SERUM SAMPLES


1



5 µl serum

Pipette 5 µl of serum into the Dilution buffer vial.
Close the vial and mix well.


2

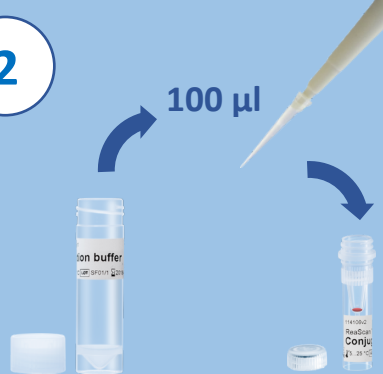

100 µl

Transfer 100 µl of diluted sample into the Conjugate vial. Mix carefully until the conjugate is completely dissolved.

3

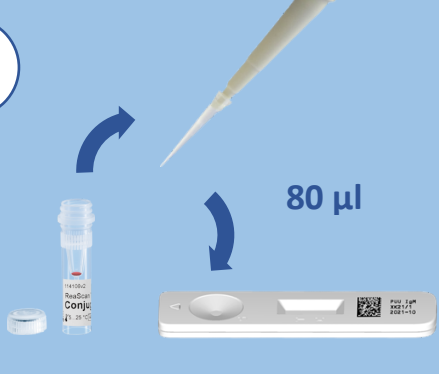

80 µl

Transfer 80 µl of the conjugate-sample mixture into the test cassette's sample well. Start the timer.

4



LYME IgM/IgG
Lot ID: YH30/2
Expiry: 2021-11-30
Sample ID: 12345

Insert test cassette and press "Start Measurement"

☒ Read Now mode
☐ Incubation mode

Start Measurement

'Read Now mode': read the result with the ReaScan+ reader within the reading time frame (18-22 minutes).
'Incubation mode': insert the cassette immediately into the ReaScan+ reader and start measurement. The test cassette will be analyzed after 20 min incubation. *Note! Do not remove the test cassette from the reader during incubation.*

5



LYME IgM/IgG
POSITIVE
12345

VisE (C6) : 78.99 Au
OspC (PepC10) : 8.08 Au
Result VisE : POS (> 10.00)
Result OspC : EQV (5.00-10.00)

2021-09-07
09:22
ADMIN

Product ID: 115103
Lot ID: YH30/2
Expiry: 2021-11-30

Print
Next sample

The result is either positive, equivocal (EQV) or negative.



Reagent

ReaScan®+ LYME IgM/IgG Combo Quick

TEST PROCEDURE FOR CSF SAMPLES

1

90 µl Dilution
buffer

10 µl
CSF



Pipette **90 µl** of Dilution buffer and **10 µl** of CSF into the Conjugate vial. Mix carefully until the conjugate is completely dissolved.

2

80 µl



Transfer **80 µl** of the conjugate-sample mixture into the test cassette's sample well. Start the timer.

3



LYME IgM/IgG
Lot ID: YH30/2
Expiry: 2021-11-30
Sample ID: 12345

Insert test cassette
and press "Start
Measurement"



Read Now mode



Incubation mode

Start Measurement

'Read Now mode': read the result with the ReaScan+ reader within the reading time frame (18-22 minutes).

'Incubation mode': insert the cassette immediately into the ReaScan+ reader and start measurement. The test cassette will be analyzed after 20 min incubation. *Note! Do not remove the test cassette from the reader during incubation.*

4



The result is either positive, equivocal (EQV) or negative.

IVD



REF 115103